 Orthofix US LLC
3451 Plano Parkway
Lewisville, Texas 75056-9453 U.S.A.
1-214-937-3199
1-888-298-5700
www.orthofix.com

Australian Sponsor
Emergo Australia
Level 20, Tower II
Darling Park
201 Sussex Street
Sydney, NSW 2000
Australia

 Medical Device Safety Service
(MDSS)
Schiffgraben 41
30175 Hannover
Germany
www.mdss.com

Device System Name:
NGage[®] Surgical Mesh System

Click directory below for desired language

English 	2-3
Español 	4-5
Italiano 	6-7
Deutsch 	8-9
Português 	10-11
Français 	12-13

Orthofix US LLC
 3451 Plano Parkway
 Lewisville, Texas 75056-9453 U.S.A.
 1-214-937-3199
 1-888-298-5700
 www.orthofix.com

Australian Sponsor
 Emergo Australia
 Level 20, Tower II
 Darling Park
 201 Sussex Street
 Sydney, NSW 2000
 Australia

EC REP Medical Device Safety Service
 (MDSS)
 Schiffgraben 41
 30175 Hannover
 Germany
 www.mdss.com

English EN

Device System Name:

NGage® Surgical Mesh System

Description:

The NGage Surgical Mesh System is a diamond pattern, surgical mesh device. The body is manufactured from commercially pure (CP) titanium conforming to ASTM F67, the end rings, the standard ring, and the screws are manufactured from titanium alloy that conforms to ASTM F136. Because of the construction, the angle and the length of the mesh can be reduced incrementally to adjust it to individual anatomical conditions. The system is sold non-sterile.

Indications for Use:

The NGage Surgical Mesh System is indicated for use in the thoraco-lumbar spine (T1-L5) to replace a diseased vertebral body resected or excised for the treatment of tumors, to achieve anterior decompression of the spinal cord and neural tissues, and to restore the height of a collapsed vertebral body.

The NGage Surgical Mesh System is also indicated for treating fractures of the thoracic and lumbar spine.

The NGage Surgical Mesh System is designed to restore the biomechanical integrity of the anterior, middle, and posterior spinal column even in the absence of fusion for a prolonged period. It is recommended to pack bone graft material inside the mesh cage prior to implantation.

The NGage Surgical Mesh System is intended for use with supplemental fixation. As an example, the supplemental fixation system that may be used with the system is the Orthofix Spinal Fixation System.

Contraindications:

The NGage Surgical Mesh System, as with other metallic orthopedic appliances, is contraindicated for use in patients with active infections in which the use of an implant could preclude adequate and appropriate treatment of the infection. The system is also contraindicated for use in patients with known or suspected metal allergies.

Potential Adverse Events:

Potential adverse events include, but are not limited to:

1. Failure of the device to provide adequate mechanical stability.
2. Loss of fixation of the implant.
3. Device component failure.
4. Migration or bending of the device.
5. Loss of bony alignment.
6. Non-union.
7. Fracture of bony structures.
8. Resorption without incorporation of any bone graft utilized.
9. Immunogenic response to the implant materials.

Note: As with any surgical procedure, there are risks involved in orthopedic surgery. Infrequent operative and postoperative complications known to occur are: early or late infection which may result in the need for additional surgeries, damage to blood vessels, spinal cord or peripheral nerves, pulmonary emboli, loss of sensory and/or motor function, impotence, and permanent pain and/or deformity. Rarely, some complications may be fatal.

Warnings and Precautions:

The surgeon should be aware of the following when using metallic implants:

1. The correct selection of the implant is extremely important. The potential for success is increased by the selection of the proper size, shape and design of the implant. The size and shape of human bones present limiting restrictions on the size and strength of implants. No implant can be expected to withstand the unsupported stresses of full weight bearing.
2. Correct handling of the implant is extremely important. Contouring the metal devices is to be avoided where possible. If contouring is necessary, it should not be bent sharply, reverse bent, notched or scratched. All of these operations can produce defects in surface finish and internal stress concentrations, which may become the focal point for eventual failure of the device.
3. Single use only. Reuse of devices labeled as single-use (e.g. implants, drills, tacks, trial rods) could result in injury or reoperation due to breakage or infection.
4. Non-sterile; the surgical mesh and instruments are sold non-sterile, and therefore, must be sterilized before each use.
5. Postoperative care is important. The patient should be instructed in the limitations of the metallic implant and should be cautioned regarding weight bearing and body stress on the device prior to secure bone healing.

6. All implants are intended for SINGLE USE ONLY. Any used implant should be discarded. Even though the device may appear undamaged, it may have small defects and internal stress patterns that may lead to fatigue failure.

MRI Compatibility Information:

The NGage Surgical Mesh System has not been evaluated for safety and compatibility in the MR environment. It has not been tested for heating, migration, or image artifact in the MR environment. The safety of NGage Surgical Mesh System in the MR environment is unknown. Scanning a patient who has this device may result in patient injury.

Cleaning:

NGage Surgical Mesh System instruments and implants are provided clean but not sterile. Once an implant comes in contact with any human tissue or bodily fluid it should not be re-sterilized or used. Please discard all contaminated implants.

All instruments must be thoroughly cleaned after each use. Cleaning may be done using validated hospital methods or following the validated cleaning processes described below.

None of the instruments in the system require disassembly prior to cleaning.

From Point of Use:

Whenever possible, do not allow blood, debris or body fluids to dry on instruments. For best results and to prolong the life of the surgical instrument reprocess immediately after use.

1. Remove excess body fluids and tissue from instruments with a disposable, non-shedding wipe. Place instruments in a basin of purified water or in a tray covered with damp towels. Do not allow saline, blood, body fluids, tissue, bone fragments or other organic debris to dry on instruments prior to cleaning.
2. For optimal results, instruments should be cleaned within 30 minutes of use or after removal from solution to minimize the potential for drying prior to cleaning.
3. Used instruments must be transported to the central supply in closed or covered containers to prevent unnecessary contamination risk.

Note: Soaking in proteolytic enzymatic detergents or other pre-cleaning solutions facilitates cleaning, especially in instruments with complex features and hard-to-reach areas (e.g. cannulated and tubular designs, etc.). These enzymatic detergents as well as enzymatic foam sprays break down protein matter and prevent blood and protein based materials from drying on instruments. Manufacturer's instructions for preparation and use of these solutions should be explicitly followed.

Preparation for Cleaning:

1. All instruments with moving parts (e.g., knobs, triggers, hinges) should be placed in the open position to allow access of the cleaning fluid to areas that are difficult to clean.
2. Soak the instruments for a minimum of 10 minutes in purified water prior to the manual or automated cleaning process.
3. Use a soft cloth or a soft plastic bristle brush to remove any visible soil from the instruments prior to manual or automated cleaning. Use a soft plastic bristle brush or a pipe cleaner to remove soil from any inner lumens. You can also use a syringe (if appropriate) for hard to reach areas.
4. Enzymatic detergent should be used for manual and automated cleaning. All enzymatic detergents should be prepared at the use dilution and temperature recommended by the manufacturer. Softened tap water may be used to prepare the enzymatic detergents. Use of recommended temperatures is important for optimal performance of enzymatic detergent.

Manual Cleaning:

1. Completely submerge instruments in an enzymatic detergent and allow to soak for 20 minutes. Use a soft-bristled, nylon brush to gently scrub the device until all visible soil has been removed. Particular attention must be given to crevices, lumens, mated surfaces, connectors and other hard-to-clean areas. Lumens should be cleaned with a long, narrow, soft-bristled brush (i.e. pipe cleaner brush).
2. Remove the instruments from the enzymatic detergent and rinse in tap water for a minimum of 3 minutes. Thoroughly and aggressively flush lumens, holes and other difficult to reach areas.
3. Place prepared cleaning solution in a sonication unit. Completely submerge device in cleaning solution and sonicate for 10 minutes.
4. Rinse instrument in purified water for at least 3 minutes or until there is no sign of blood or soil on the device or in the rinse stream. Thoroughly and aggressively flush lumens, holes and other difficult to reach areas.
5. Repeat the sonication and rinse steps above.
6. Remove excess moisture from the instrument with a clean, absorbent and non-shedding wipe.
7. Inspect the instruments for visible soil.
8. If visible soil is noted, repeat the steps listed above.

Automated Cleaning:

1. Completely submerge the instruments in an enzymatic detergent and allow to soak and sonicate for 10 minutes each. Use a soft nylon bristled brush to gently scrub the device until all visible soil has been removed. Particular attention must be given to crevices, lumens, mated surfaces, connectors and other hard to clean areas. Lumens should be cleaned with a long, narrow, soft nylon bristled brush (i.e. pipe cleaner). Use of a syringe or waterjet will improve flushing of difficult to reach areas and closely mated surfaces.
2. Remove instruments from the cleaning solution and rinse in purified water for a minimum of 1 minute. Thoroughly and aggressively flush lumens, blind holes and other difficult to reach areas.
3. Place instruments in a suitable washer/disinfector basket and process through a standard instrument washer/disinfector cleaning cycle.
4. Orient instruments into the automated washer's carriers as recommended by the washer manufacturer.
5. The following minimum parameters are essential for thorough cleaning.
 - a. 2 minute prewash with cold tap water
 - b. 1 minute prewash with hot tap water
 - c. 2 minute detergent wash with hot tap water (64-66°C/146-150°F)
 - d. 1 minute hot tap water rinse
 - e. 2 minute thermal rinse with purified water (80-93°C/176-200°F)
 - f. 1 minute purified water rinse (64-66°C/146-150°F)
 - g. 7 to 30 minute hot air dry (116°C/240°F)
6. Inspect the instruments for visible soil.
7. If visible soil is noted, repeat the above listed steps until no visible soil is noted.

Note: Certain cleaning solutions such as those containing caustic soda, formalin, glutaraldehyde, bleach, and/or other alkaline cleaners may damage instruments. These solutions should not be used.

Note: Visually inspect instruments after cleaning and prior to each use. Discard or return to Orthofix any instruments that are broken, discolored, corroded, have cracked components, pits, gouges, or are otherwise found defective. Do not use defective instruments.

Instrument End of Life Determination:

Do not reuse Single Use instruments. Visually inspect the reusable instruments to determine if the instrument has reached end of life. Orthofix reusable instruments have reached End of Life when:

1. Instruments show signs of damage such as binding, bending, breakage, overt signs of wear and/or any other conditions which may impact the device safe and effective use.
2. Instruments intended for cutting bone and/or tissue (e.g. tap, rasp, curette, rongeur) – when any of the cutting surfaces show signs of wear such as nicks, abrasions or otherwise dulled cutting surfaces.
3. Instruments that interface with other devices (e.g. implants, instruments, handles) - when the mating feature binds, fails to attach or fails to hold the device securely. The instrument function should be verified prior to each use.
4. Do not use instruments which reached End of Life. Discard End of Life instruments per your hospital procedure or return to Orthofix for disposal.

Sterilization:

The NGage Surgical Mesh System components are supplied NON-STERILE. Prior to use, all components should be steam sterilized by the hospital using the recommended cycle:

Method: Steam	or:	Method: Steam
Cycle: Gravity		Cycle: Prevac
Temperature: 250°F (121°C)		Temperature: 270°F (132°C)
Exposure time: 30 minutes		Exposure time: 8 minutes

Packaging:

Packages for each of the components should be intact upon receipt. If a consignment system is used, all sets should be carefully checked for completeness and all components should be carefully checked for damage prior to use. Damaged packages or products should not be used and should be returned to Orthofix.

The NGage Surgical Mesh System instruments and implants are provided in modular cases specifically intended to contain and organize the system's components. The system's instruments are organized into trays within each modular case for easy retrieval during surgery. These trays also provide protection to the system components during shipping. Additionally, individual instruments and implants are provided in sealed poly bags with individual product labels.

Product Complaints:

Any Healthcare Professional (e.g., customer or user of this system of products) who has any complaints or who has experienced any dissatisfaction with the product quality, identity, durability, reliability, safety, effectiveness and/or performance, should notify Orthofix US LLC, 3451 Plano Parkway, Lewisville, TX 75056, USA, by telephone at 1-214-937-3199 or 1-888-298-5700 or by e-mail at complaints@orthofix.com.

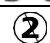
Further Information:

A recommended operative technique for the use of this system is available upon request from Orthofix at the phone numbers provided above.

Latex Information:

The implants, instruments and/or packaging material for the NGage Surgical Mesh System are not formulated with and do not contain natural rubber. The term "natural rubber" includes natural rubber latex, dry natural rubber, and synthetic latex or synthetic rubber that contains natural rubber in its formulation.

Caution: Federal law (USA) restricts these devices to sale by or on the order of a physician.

Rx Only		Federal (U.S.A.) law restricts this device to sale by or on the order of a physician	
	See Instructions for Use		Manufacturer
	Orthofix.com/IFU		Authorized Representative
	Single Use Only Do Not Reuse		Lot Number
	Provided Non-Sterile		Catalogue Number

Orthofix US LLC
3451 Plano Parkway
Lewisville, Texas 75056-9453 U.S.A.
1-214-937-3199
1-888-298-5700
www.orthofix.com

Australian Sponsor
Emergo Australia
Level 20, Tower II
Darling Park
201 Sussex Street
Sydney, NSW 2000
Australia

Medical Device Safety Service
(MDSS)
Schiffgraben 41
30175 Hannover
Germany
www.mdss.com

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