

Fitting & Care Instructions - Correx TLSO

Model: DCT-5657

Instructions for putting on brace



Step 1: Open brace and lay flat on a table. Detach side wings from right and left to match the patients waist circumference and size.



Step 2: Loosen shoulder straps and slide on to patient like a book bag.



Step 3: Wrap side wings around waist and Velcro closed around the patient's abdomen.



Step 4: Tighten shoulder straps by pulling down on straps.



Step 5: Tighten waist belt by pulling on lower pull tabs to desired support.



Step 6: Locate chest support buckle and adjust height by detaching it and then reattaching to shoulder straps. Once correct height is found, tighten by pulling on straps on either side.



Step 7: Find upper pull tabs connected to shoulder straps. Pull to desired extension.



Step 8: (Optional) The thoracic pull tabs and lumbar pull tabs can be attached to each other to use as a single pull tab.

Care Instructions: Hand wash with a light detergent and let air dry.

Warranty: 6 month warranty from any manufacturer defects

Step 9: Instruct the patient in the application and removal of the device and observe the patient apply and remove the brace at least one time.

Size 0	Size 1	Size 2	Size 3	Size 4	Size 5	Size 6	Size 7
22" - 32"	31" - 37"	36" - 40"	39" - 43"	42" - 46"	45" - 49"	48" - 52"	51" - 55"