

Fitting & Care Instructions - Extend LSO

Model: DCT-3951

Instructions for putting on brace



Step 1: Remove lateral panels and lay brace on a table facing up. Find your size by referencing the sizing chart.



Step 2: Remove overlapping front panel on both right and left sides.



Step 3: When completely detached, your brace will look like the above image.



Step 4: Find your size and attach to both right and left side.



Step 5: Once sizing is completed, reattach the lateral panels and center the back panel.



Step 6: Overlap right side wing and attach to left side wing for a snug fit.



Step 7: Locate rubber pull tabs at the end of the right and left string and pull to desired support.



Step 8: Once desired support is achieved, attach the right pull tab and left pull tab to the front of the brace.

Care Instructions: Hand wash with a mild soap and let air dry.

Warranty: The DCT-3951 is covered by a 5 year warranty against all mechanical or manufacturer defects from the date of purchase to original owner.

Size 0	Size 1	Size 2	Size 3	Size 4	Size 5	Size 6	Size 7
22" - 32"	31" - 37"	36" - 40"	39" - 43"	42" - 46"	45" - 49"	48" - 52"	51" - 55"

Above measurement is based on waist circumference. There is an option for a 10" extension panel for patients larger than 55".