



Fitting and Care Instructions - ActivAIR
Tall/Short Walking Boot

Model: DCT-6061T/DCT-6061S

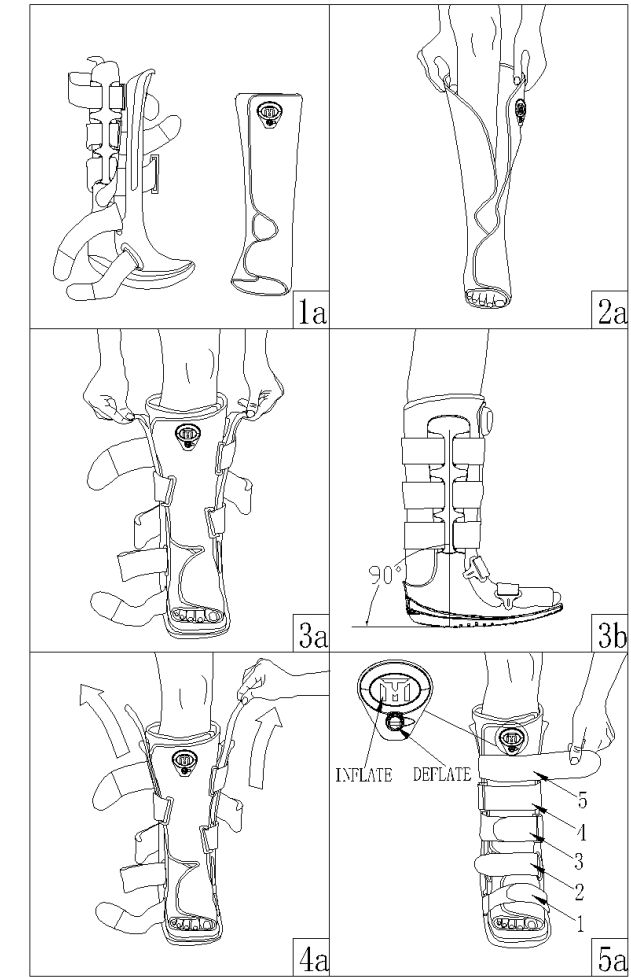


Image showing: DCT-6061T

Thank you for purchasing our products! Please read this instruction carefully before operation! To fully understand the product, in order to ensure the safety and effectiveness of the product, and please properly save the manual for future reference.

Product Overview

Product Name: ActivAIR Tall/Short Walking Boot

Product Model: DCT-6061T/DCT-6061S

Product Structure: The product is mainly composed of base, strut, strap, lining, adhesive, and bladder.

Operating Principle: The main body of the product is equipped with support, non-skid soft sole and bladder. The ankle joint and calf are properly protected by support device. Through the soft materials and shock absorption systems, patients have more comfort in their legs during exercise. The combination of the upper body of the shoe and the front cover of the calf makes the leg of the patient well protected. The breathable soft insole makes the product provide comprehensive protection to the foot before and after the use of the foot, prevent secondary injury, and provide soft contact between the calf and the fixed support to make it more comfortable to walk. The combination of soft shock absorber and anti-skid soft sole can give the patient's foot a good cushioning effect and help the patient get a good recovery.

Scope of Application: External use, provides compressed air support for foot and/or ankle stability fractures, severe ankle sprains, and postoperative rehabilitation for fracture heals and edema (swelling) treatment.

Product Features

Product Features and Storage Conditions

Product Features: New generation air pump system, low profile rocker sole, shock-absorbing technology, high strength and light weight mechanical strap locking system.

Storage Conditions: This product should be stored in a room with no more than 80% relative humidity, no corrosive gas and good ventilation to avoid high temperature.

Application Instructions

Note 1: Do not contact the liner directly with the skin.

Please purchase and use the product under the guidance of your doctor or professional.

STEP 1: Release the nylon buckle and remove the internal gasket (Fig 1a).
STEP 2: Place the foot in the pad, heel close to the back of the pad, wrap the foot and leg. Make sure the legs and feet are close to the gasket (Fig 2a).
STEP 3: Gently expand the side pillar and place the calf n the bracket while the pull strip is intact. The ankle should be concentrated in the pillar, with the foot and ankle at 90° (Fig 3b).
STEP 4: After correct alignment, remove the pull strip, gently push the pillar onto the gasket, and combine the buckle on the gasket (Fig 4a).
STEP 5: Fasten the strap 1-2-3-4-5 (1-2-3 for DCT-6061S) from the bottom to the top of the toe, pay attention to the hole position of the strap and the column position on the support frame can not be separated, fasten the strap but not too tight. As the swelling pattern changes in one day, it may be desirable to adjust the degree of tightness of the strap for comfort. After fixation, inflate the liner press and release the central bubble ball until the liner provides comfortable cushioning and support. Do not overfill. To deflate or adjust the pressure, press the smaller circular side button. (Fig 5a).
STEP 6: To remove press the release button until the air is finished and then loosen the straps. Pull the heel and ankle forward. If you want to wear again, put the foot and ankle in the pad and fix the straps again.

Note 2:

1. Do not inflate the airbag too much. Inappropriate balloon inflation may cause significant stimulation in patients with diabetic neuropathy. If any visible skin changes occur or the patient reports discomfort, reduce the pressure on the airbag. Do not use this instrument for patients who are unable to express discomfort.
2. When walking on a slippery surface, care should be taken to avoid injury. At high altitude, the expansion of airbags will exceed the optimal level.
3. Be sure to examine product before use. Do not use damaged products. If there is any tear or wear on the instrument, please stop using it.

Maintenance

The soft fabric part may be washed with a neutral detergent in warm water up to 30°C and air-dried. Do not put the dryer or use other heat sources for drying. Elements such as salt water, sand, dirt, etc. may cause injury. After exposure to these conditions, rinse thoroughly with clear water, dry the towel or air-dry (do not machine the machine). For detergents, use mild detergent and do not use softeners and bleaches. The rigid part can be scrubbed with wet cloth and dried naturally.

Warning

If you experience pain, swelling, irritation, or discomfort while using this product, discontinue use and consult your doctor.
This product is for support only and is not intended to prevent injury or to prevent the recurrence of injuries.

Basic Product Environment

- 1) Ambient temperature: -15 ~ 60°C
- 2) Relative humidity: 80%

Product Material

- 1) Base/Strut: Polypropylene + Glass Fiber
- 2) Straps: Nylon
- 3) Pad: Nylon / Polyurethane / Polyester (outer net / foam / inner mesh)
- 4) Bladder/pump: Thermo Plastic Polyurethane
- 5) Shoe Outsole/Shoe Inner Sole: Thermoplastic Elastomer/ Polypropylene/EVA

Contraindication

- ⚠ Use with caution for unstable fractures
- ⚠ Use with caution in osteoporosis
- ⚠ Use with caution for cognitive impairment
- ⚠ Use with caution for bony ankylosis

Storage, Transportation and Expiry Date of Products

- Product is valid: Five years
- Product storage: Please place it in a dry and clean room before use to ensure ventilation
- Transportation: The product shall be dust-proof, moisture-proof, and pollution-proof during storage and transportation

Disposal of wastes

Disposal of wrapping paper (bags): please dispose of the packaging of this product according to the local laws.

Symbol and picture description

	Manufacture Date		Batch Code
	Medical Device		CE Symbol
	Hand Wash Only		Protect From Sunlight
	Do Not Use When The Packing Is Damaged		
	Warnings and Precautions		
	Reference Instructions		

Accessories List

Instructions ----- 1 copy
Pad ----- 3 pcs

After-sales Warranty

Warranty: all products from the date of purchase as the original date, normal wear and tear of the product in the use of the process is not considered defective.
The following parts from the date of purchase to the warranty period, if there is any material or process defects, please contact your seller directly.

Please refer to the Instructions before usage

	Manufactured by: Trend Medical LLC 6583 Ruch Rd. Bethelhem, PA 18017
	Distributed by: ORTHOFIX Phone: 800-535-4492 Email: PEMFCustomerService@Orthofix.com www.Orthofix.com/Bracing