

Fitting and Care Instructions - Air/Gel Ankle Stirrup

Model: DCT-4350

Instructions for putting on brace



Step 1: Place the arch of your foot at the bottom of the footplate.



Step 2: Locate the top strap and wrap it around to the opposite side. Tighten the strap for a secure fit.



Step 3: Locate the bottom strap and wrap it around to the opposite side. Tighten the strap for a secure fit.



Step 4: This brace has the option to place the gel pads in the refrigerator for cold therapy. To do so, remove the bladders from either side of the plastic frame and place them in the refrigerator. Once cold, place the bladders back into the splint and complete Steps 1-3.

Care Instructions: Hand wash with a light detergent and let air dry.

Warranty Information: 6 Month Warranty against any manufacturing defects.