


Fitting and Care Instructions - Pro Wrist

Model: DCT-3916

Instructions for putting on brace

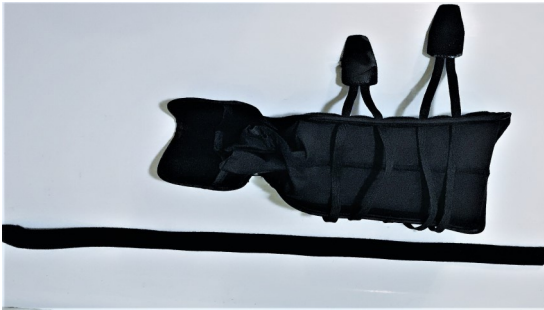

Step 1: Detach removable strap, unlace pull tabs and gently bend mold-able aluminum arms open.



Step 2: Un-Velcro universal sizing strip and slide TOWARD elastic neoprene to increase wrist circumference, or slide AWAY from elastic neoprene for smaller wrist circumference.



Step 3: Insert arm and gently, but firmly, mold aluminum arms until the fit snug on the forearm, wrist, and hand.



Step 4: Pull each PULL TAB directly across the brace and Velcro to the back side, this will give you your compression of the forearm and wrist.



Step 5: Twist dial on backside of wrist to the LEFT in order to loosen the range of motion setting, bend wrist foreword or back to desired setting, twist dial to the RIGHT to tighten and hold patient hand in place.





Step 6: Velcro strip attaches at wrist, just to the side of the dial, pull strap across the BACKSIDE of the hand and between the THUMB and POINTER FINGER.



Step 6 continued: Strap pulls across the UPPER PALM AREA and around the back. Velcro the strip to the brace where it feels best (will vary due to size of patient arm).

Warranty: The Pro Wrist has a 6 month warranty against any manufacturing defects from the date of purchase.

Care Instructions: Hand wash with a light detergent and let air dry.

Universally Sized Ambidextrous