

Fitting and Care Instructions - QuickForm Ankle

Model: DCT-1902

Instructions for putting on brace



Step 1: Open the brace and slide your foot into it, with the mesh pad positioned on top of your foot.



Step 2: Locate the pull tab on the brace and tighten to the desired level of support.



Step 3: Locate the straps on the side of the brace, wrap them underneath your foot and attach them securely.



Step 4: Wrap both straps under the foot and attach them to the opposite.



Step 5: Locate the top compression strap and wrap it around your ankle to the desired level of support.

Care Instructions: Hand wash with a light detergent and let air dry.

Warranty Information: 6 Month Warranty against any manufacturing defects.