

Fitting and Care Instructions - Universal Wrist and Thumb

Model: DCT-200

Instructions for putting on brace



Step 1: Unstrap brace and slide injured hand into sleeve with the palm down.



Step 2: Move large Velcro platform for tighter fit if needed, tighten and adhere top and bottom strap to desired support.



Step 3: Wrap inner thumb strap around thumb and adhere to opposite side.



Step 4: Wrap outer thumb strap and adhere to opposite side.



Step 5: Tighten thumb strap around thumb at desired support.

Care Instructions:

Hand wash with mild detergent and air dry.

Warranty:

The Universal Wrist & Thumb is covered by a 6 month warranty against all mechanical or manufacturer defects, from the date of purchase.

Universally Sized Right/Left