

Fitting and Care Instructions - Wrist Extension

Model: DCT-908

Instructions for putting on brace



Step 1: Unstrap brace and slide injured hand into sleeve with the palm down, adjust the sizing platform front or back depending on wrist circumference.



Step 2: Locate strap near the thumb and wrap around thumb and attach to desired support.



Step 3: Locate upper strap and tighten to desired support.



Step 4: Locate bottom strap and wrap to desired support.



Step 5: Double check all straps to ensure a proper fit.

Care Instructions:

Hand wash with mild detergent and air dry.

Warranty:

The Wrist Extension is covered by a 6 month warranty against all mechanical or manufacturer defects, from the date of purchase.

Universally Sized Right/Left